# HOME QUARANTINE MANAGEMENT SYSTEM

Powered by

A member of UEM Group

## WHAT IS HOME QUARANTINE MANAGEMENT SYSTEM?

### APAKAH HOME QUARANTINE MANAGEMENT SYSTEM?

The Home Quarantine Management System (HQMS) is integrated with GPS technology for real-time monitoring of individuals who are required to undergo home quarantine as stipulated by the Ministry of Health Malaysia.

It consists of an **RFID** (Radio-frequency Identification) e-Bracelet and **CTU** (Covert Tracking Unit) which will trigger alerts and notify authorities of any violations.

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# HOW HQMS WORKS?

## BAGAIMANA HQMS BERFUNGSI?

HQMS is monitored 24/7 via QuickMed, our smart system. Any quarantine violations including trespassing beyond the geofencing area made by Person Under Investigation (PUI) / Person Under Surveillance (PUS) will be recorded and notified to the authorities.

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## HOME SURVEILLANCE ORDER PERINTAH PENGAWASAN DAN PEMERHATIAN DI RUMAH

1. You will be given Home Surveillance Order (HSO) devices which consists of a digital wristband (RFID e-Bracelet) and CTU.  
*Anda akan diberi alat-alat Perintah Pengawasan dan Pemerhatian Di Rumah (HSO) yang terdiri daripada gelang tangan digital (e-Bracelet RFID) dan CTU.*
2. You must remain at the stated / registered address during your entire quarantine period.  
*Anda harus kekal berada di alamat yang dinyatakan / didaftar sepanjang tempoh kuarantin anda.*
3. Wear a face mask, always practice physical distancing and comply with SOPs during your quarantine period.  
*Sentiasa pakai pelitup muka, amalkan penjarakan fizikal dan patuhi SOP semasa tempoh kuarantin anda.*
4. You are not allowed to receive any visitors throughout the quarantine period.  
*Anda tidak dibenarkan menerima pelawat sepanjang tempoh kuarantin.*
5. If you have symptoms, please seek treatment at a nearby healthcare facility.  
*Jika anda mengalami sebarang gejala, sila dapatkan rawatan di klinik / hospital yang berdekatan.*
6. Permission from the nearest District Health Office is required before leaving your designated quarantine location for any clinic / hospital appointments.  
*Kebenaran daripada Pejabat Kesihatan Daerah terdekat diperlukan sebelum meninggalkan lokasi kuarantin yang ditetapkan untuk sebarang janji temu klinik / hospital.*
7. You are required to obtain a Release Order and remove the RFID e-Bracelet on Day-8 for Person Under Investigation (PUI) / Person Under Surveillance (PUS) and Day-11 for confirmed positive patients.  
*Anda dikehendaki mendapatkan Perintah Pelepasan dan memulangkan e-Bracelet RFID pada hari ke-8 bagi individu dalam siasatan (Patient Under Investigation, PUI) / individu dalam pengawasan (Patient Under Surveillance, PUS) dan hari ke-11 bagi pesakit yang disahkan positif.*

# FAQs

---

## 1. Why do I need to wear the RFID e-Bracelet (Digital Tracker)?

Despite being fully vaccinated, there is still a small probability that we can be infected with the virus. Self-quarantine will ensure to minimise the transmission of the COVID-19 virus

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## 2. My e-Bracelet is blinking, what should I do?

It is normal for the e-Bracelet to blink once in a while for a few seconds for data transferring signal.

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## 3. The CTU blinks “Red” and “Green” lights. What does that mean?

The blinking “Red” and “Green” lights on the CTU indicates that the device is sending a signal and information to the system.

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## 4. What should I do if I have issues with my e-Bracelet?

Kindly contact us using the provided channels below as soon as possible to resolve the issue.

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## 5. Is the e-Bracelet water-resistant?

Yes, the e-Bracelet is water-resistant.

### ***Adakah e-Bracelet ini kalis air?***

*Ya, e-Bracelet tersebut kalis air.*

## 6. Do we need to charge the e-Bracelet?

The e-Bracelet is not required to be charged during your quarantine period.

### ***Adakah e-Bracelet perlu di caj?***

*e-Bracelet tidak perlu di caj sepanjang tempoh kuarantin anda.*

## 7. What should I do on my 5<sup>th</sup> day of home quarantine?

You are required to do your second PCR swab test on the 5<sup>th</sup> day of home quarantine. Please proceed to the nearest government / private hospital, government / private clinic, or any medical service provider to take the test. Once completed, please return to your quarantine address immediately. You **do not** have to call in and inform us.

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## 8. What should I do after finishing my home quarantine?

You are required to return the devices at our available removal counters nearest to you. No appointment booking is required.

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## 9. Where should I return the devices?

You need to return the devices at the following locations upon the completion of your quarantine.

### **Klang Valley:**

UEM Edgenta Learning Centre,  
Jalan 51/217, Off Jalan Templer,  
Petaling Jaya, 46050, Petaling Jaya,  
Selangor

### **Removal Counter Operating Hours:**

9:00am - 5:00pm daily, including weekends and public holidays

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Please scan the QR code for more details and list of locations for removal counters nationwide.

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